

Tonsil Fire Extinguisher Instructions for Use

For Rx use only. Federal law restricts device to sale by or on the order of a physician or other authorized medical practitioner.

INTENDED USE: This device is intended specifically to administer medicinal substances to treat throat disorders. This medication should only be used in the amount and dosage timing as directed by your physician. More frequent use or dosage above the recommended amounts can result in harmful effects.

1. Remove clear cap.
2. Extend spraying arm and hold pointing toward you (NOTE: arm must be at right angle for proper deployment).
3. Hold the bottle firmly with forefinger on spraying head.
4. Aim arm directly at desired area, getting tip of sprayer as close as comfortably possible and try to keep the tongue down.
5. Firmly press plunger to dispense spray – DO NOT BREATHE IN WHILE SPRAYING!

SINGLE PATIENT USE

Rx Only

RxOnly

LOT

Batch code

REF

Catalog number



Caution



Do not use if package is damaged




Non-sterile



Electronic Instructions for use

eIFU Indicator

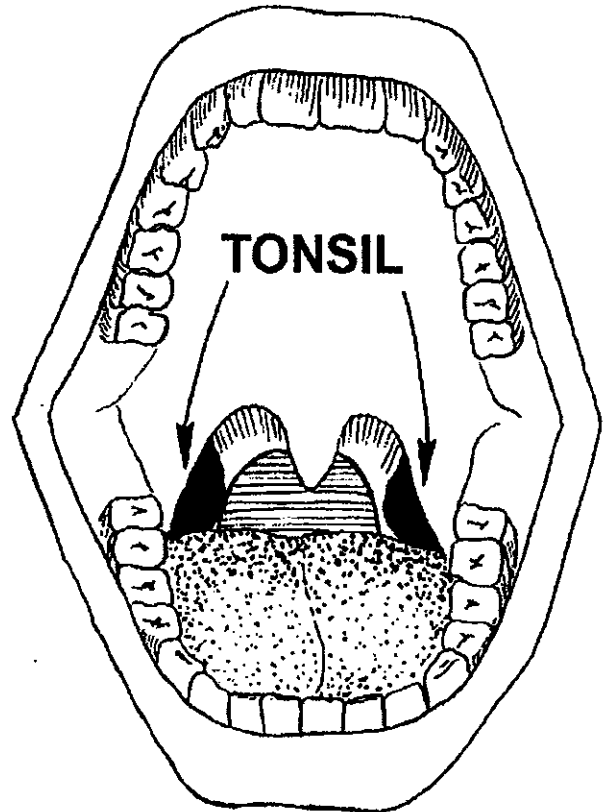

CE



Brilliant, LLC
888 E Belvidere Road, Suite 212
Grayslake, IL 60030 USA

EC REP

mdi Europa GmbH
Langenhagener Straße 71
D-30855 Langenhagen, Germany



CAUTION: Not for use in children under 10 years of age.

BR-LS-0001(Rev 1)2017-9_US